

CLEAN CERTIFICATION

Important: This form must be filled out and faxed to Buxton before returning any medical device to our facility, regardless of actual use in Surgery.

FACILITY: _____

☐ Loaner/Rental ☐ Repairs ☐ Returns

P.O./Invoice/Ref: _____

Item: _____

CERTIFICATION

We certify that the instruments planned for shipment to Buxton have been cleaned and sterilized for safe handling.

1. Clean and disinfect instrument immediately after use. Do not allow biofluids or other material to dry on the instrument; wipe clean and keep moist until the instrument reaches sterile processing. Rinse thoroughly in lukewarm water. Carefully scrub parts with soft brushes. Flush lumens and joints using a syringe filled with demineralized water. Use low-foaming, enzymatic cleaning solutions with a pH range of 6-8 and follow manufacturer's instructions. Rinse with demineralized water. Lubricate in a water-soluble instrument milk bath.

Please initial

2. Sterilize wrapped devices using pre-vacuum cycle for four minutes at 270°F (132°C). Due to the potential damage to non-metal components, chemical sterilization and hot air or steam sterilization in excess of 273°F (134°C) should be avoided.

Please initial

IN CASE OF NON-USE: We certify these instruments were neither used in surgery nor came in contact with any contaminated device, and therefore do not require cleaning and sterilization.

Please initial

☐ Product never opened ☐ Product opened, but reviewed only outside of a surgical or laboratory environment. ☐ _____

Sterile Processing Signature

Printed Name and Title

Date

Phone: _____ Fax/email: _____

TO OBTAIN A RETURN AUTHORIZATION NUMBER:

COMPLETE this form and FAX to 973-215-2922.
Or scan and email to info@buxtonbio.com

When received, Buxton will return this form by your choice of fax or email with the assigned RMA number. Make a copy for your records and fold the other copy, placing it in a packing slip pouch. Attach it to the outside of the box, making sure the "Return to:" address (shown to the right of these instructions) and the RMA number can be seen through the pouch. Any packages sent back to our facility **without a signed cleaning certification and return authorization number** are subject to refusal or a \$500 cleaning fee.

Any questions? Please call us at 1-800-427-5351.

Fold this page with this panel showing on the outside of the box.

For office use

Return Authorization number (good for 30 days):

Return to:
Buxton BioMedical, Inc.
11 Melanie Lane
East Hanover, NJ 07936
Attn: Returns

fold